**HARTMANN**

# Cosmos<sup>®</sup> hydro

Hydrocolloid plasters for blisters 100 %  
Hydrocolloid\*

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## EN Instructions for use

### Product description

The hydrocolloid plaster is a non-sterile, single-use, self-adhesive plaster with a 100 % hydrocolloid\* formulation and occlusive properties (moist blisters treatment). Plasters are cushioning, waterproof.

### Composition

The plaster consists of one layer of adhesive hydrocolloid material (containing carboxymethyl-cellulose). The plaster is made of 100 % hydrocolloid\*. The silicone paper protects the coated side of the plaster.

### Properties

Whilst traditional plasters work via dry wound healing, which allows the covered tissue to dehydrate and dry out, the hydrocolloid plaster is based on the mode of action of moist healing (occlusive properties). Self-adhesive hydrocolloid plasters contain a layer which absorbs exudate and expands into a gel-like mass, thus providing cushioning for the treated damaged area and helping to relieve pressure and friction.

### Intended use

Hydrocolloid plasters are non-active, single-use, self-adhesive, non-sterile devices consisting of a self-adhesive hydrocolloid contact layer and an adhesive coated backing material (carrier material), protected with a silicone paper, for the protection and treatment of blisters and absorption of exudates. The products are intended for humans, without restrictions (in age group, weight range, health status or condition), for use by lay users, in contact with intact skin (healthy skin) and breached skin. The products do not contain resorbable components, medicinal products/ drug substances (derived from human blood or plasma or not), tissues or cells of human or animal origin, radioactive substances, nanomaterials or other hazardous components.

### Purpose

Cosmos<sup>®</sup> hydro hydrocolloid plasters for blisters is designed to effectively protect blisters from contamination, wetting, exposure to the external environment, bacteria, maintain a natural level of moisture, absorb exudate from calluses, reduce pain, create optimal conditions for healing, and also to prevent the formation of blisters.

### Indication for use

Protection of blisters from contamination, wetting, exposure to the external environment, and bacteria. Healing of blisters. Maintaining a natural level of moisture. Absorption of exudate.

### Contraindications

Untreated infected blisters, untreated infected minor (superficial) skin lesions, serious injuries, severely bleeding or profusely exuding wounds, bedsores, or for any purpose other than those listed in the intended use, individual intolerance to manufacturing materials.

### Possible side effects

Allergic reaction to the materials used to make the product.

### Product disposal

The disposable components and used plasters can be disposed of normally with household waste.

### Mode of application

- Wash hands and ensure that the affected area is clean, dry and oil-free before applying the plaster.
- Open the package and pull out the plaster.
- Cut the plaster to the required size.
- Warm the plaster with the hands before applying.
- Remove the first protective paper and place the exposed part of the plaster to the affected skin area.
- Then gently pull the second protective paper and apply the rest of the plaster. Gently mould the plaster edges to ensure optimum adhesion.
- The application maximum of the hydrocolloid plaster is up to 4 days per individual product. Should you wish to remove the plaster early or at the maximum time of application, first wet it briefly with warm water.

## Special precautions

- For single use only. Not reusable.
- Only apply the plaster if the hydrocolloid pad completely covers the blister or minor wound.
- Consult a physician before applying the plaster on a potentially infected blister or because signs of infection (eg, redness, heat, swelling, or pain) may appear.
- In cases of diabetes mellitus always consult a doctor before using plasters.
- Do not burst a sealed blister as this could risk an infection.
- Do not leave the product on the affected area for more than 4 days.
- Store the plasters in a cool (not refrigerated) and dry place at a temperature of +5 °C to +30 °C, keep out of the reach of children.

## Incident reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

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